
# User manual

Bathroom scale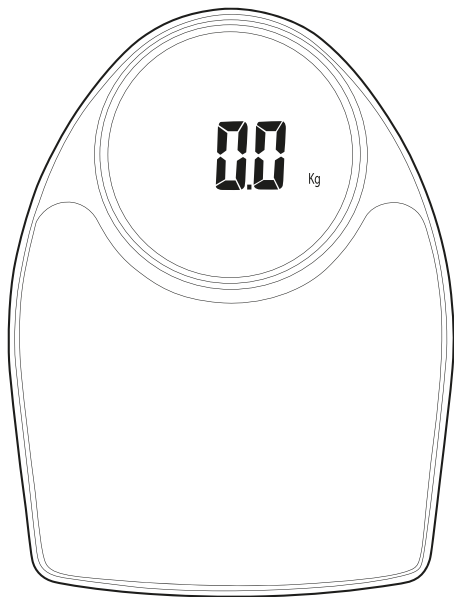

**EN** | Please retain instructions for future reference.

## SAFETY INSTRUCTIONS

**BATTERY WARNING: KEEP OUT OF THE REACH OF CHILDREN.**

- This product is not a toy.
- Children must be supervised to ensure that they do not handle or play with the product.
- This product contains no user serviceable parts. Only a qualified electrician should attempt repairs.
- If any part of the appliance is malfunctioning or if it has been dropped or damaged, cease using the product immediately to avoid potential injury.
- Do not insert fingers into any openings or inside the product.
- Do not place, store or use the product near direct sunlight or direct heat sources or in a damp/high humidity environment.
- This product is a high precision device; handle it with care.
- This product is intended for domestic use only.

## BATTERY SAFETY

- This product is supplied with 3 x AAA batteries. Only batteries of the same or equivalent type as those recommended should be used with this product.
- Batteries are to be inserted with the correct polarity.
- Exhausted batteries should be removed from the product to avoid leakage.

## Care and maintenance

To clean, wipe the exterior with a soft, damp cloth and allow to dry thoroughly.

Do not immerse the scale in water or any other liquid. Never use harsh or abrasive cleaning detergents or scourers to clean the scale, as this could cause damage.

## Before first use

Open the battery compartment on the underside of the scale.

Remove the plastic isolator tab from the battery

compartment (if present) prior to use.

Make sure the batteries are inserted following the correct polarity.

Position the scale on a firm flat surface.

## Changing the unit

**STEP 1:** Press the centre of the platform to turn on the scale.

**STEP 2:** Select kg, lb or st weight mode using the switch on the underside of the scale. The scale will then turn off.

## Zero setting the scale

**STEP 1:** Press the centre of the platform to turn on the scale, then make sure no weight is on the scale. '0.0/ZERO' will be displayed.

**STEP 2:** The scale will switch off and is now ready for use.

**NOTE:** The zero-setting process must be repeated if the scale is moved. At all other times step straight on the scale.

## Using the bathroom scale

**STEP 1:** Step on to the platform and stand still.

**STEP 2:** After 2–3 seconds the weight reading will be displayed.

**STEP 3:** Step off the platform; the weight reading will be displayed for several seconds before the scale turns off.

**NOTE:** It is recommended that readings are taken at the same time of day, preferably early evening and before a meal for the most consistent results.

## Troubleshooting

### If the scale is not working properly:

Check the batteries are fitted correctly.

Check that the correct unit is selected.

Make sure the scale is on a firm, flat surface and not touching a wall.

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Batteri: 3 x AAA-batterier (ingår)

Enhet: kg/lb/st

**EN | Disposal of Waste Batteries and Electrical and Electronic Equipment**  
This product is labelled with this crossed out wheel bin symbol in accordance with European Directive 2012/19/EC to indicate that it must not be disposed of with your other household waste. Due to the presence of hazardous substances, mixtures or components, electrical and electronic devices that are not subject to selective sorting are potentially dangerous to the environment and human health. Please check your local city office or waste disposal service for the return and recycling of this product.

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Ultimate Products UK Ltd.,
Ultimate Products Europe Ltd.,