
User manual

Bathroom scale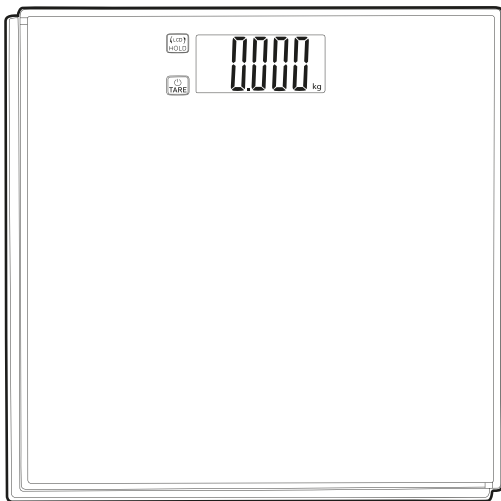

EN | Please retain instructions for future reference.

SAFETY INSTRUCTIONS

BATTERY WARNING: KEEP OUT OF THE REACH OF CHILDREN.

- This product is not a toy.
- Children must be supervised to ensure that they do not handle or play with the product.
- This product contains no user serviceable parts. Only a qualified electrician should attempt repairs.
- If any part of the appliance is malfunctioning or if it has been dropped or damaged, cease using the product immediately to avoid potential injury.
- Do not insert fingers into any openings or inside the product.
- Do not place, store or use the product near direct sunlight or direct heat sources or in a damp/high humidity environment.
- This product is a high precision device; handle it with care.
- This product is intended for domestic use only.

BATTERY SAFETY

- This product is supplied with 3 x AAA batteries. Only batteries of the same or equivalent type as those recommended should be used with this product.
- Batteries are to be inserted with the correct polarity.
- Exhausted batteries should be removed from the product to avoid leakage.

Care and maintenance

Wipe the exterior of the scale with a soft, damp cloth and allow it to dry thoroughly.

Never immerse the scale in water or any other liquid.

Do not use harsh or abrasive cleaning detergents or scourers to clean the scale, as this could cause damage.

Before first use

Remove the plastic isolator tab from the battery compartment (if present) prior to use.

Make sure the batteries are inserted following the correct polarity.

Position the scale on a firm flat surface.

Changing the unit

STEP 1: Press '⏻/TARE' to turn on the scale.

STEP 2: Select kg, st or lb weight mode using the button on the underside of the scale. The scale will then turn off.

Using as a baby scale

STEP 1: Slide the baby weighing tray onto the scale.

STEP 2: Press '⏻/TARE' to turn on the scale.

STEP 3: Tap 'LCD/HOLD' to switch the scale to baby scale mode.

STEP 4: Place the baby on the centre of the weighing tray.

STEP 5: After 2–3 seconds the weight reading will be displayed.

STEP 6: If a stable weight reading cannot be obtained, press 'LCD/HOLD' to stop the weight from fluctuating.

STEP 7: Remove the baby from the platform; the weight reading will continue to be displayed for several seconds.

Using as a bathroom scale

STEP 1: Press '⏻/TARE' to turn on the scale.

STEP 2: Tap 'LCD/HOLD' to switch the scale to the bathroom scale mode.

STEP 3: Step on to the platform and stand still.

STEP 4: After 2–3 seconds the weight reading will be displayed.

STEP 5: Step off the platform; the weight reading will be displayed for several seconds before the scale turns off.

NOTE: It is recommended that readings are taken at the same time of day, preferably early evening and before a meal for the most consistent results.

Troubleshooting

If the scale is not working properly:

Check the batteries are fitted correctly.

Check that the correct unit is selected.

Make sure the scale is on a firm, flat surface and not touching a wall.

Warning indicators

Lo – Replace battery.

EEEE – Weight exceeds maximum capacity.

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Enheter för babyvåg: kg/lbs/lb:oz

Enheter för badrumsvåg: kg/lbs/st:lb

**EN | Disposal of Waste Batteries and Electrical and Electronic Equipment**

This product is labelled with this crossed out wheel bin symbol in accordance with European Directive 2012/19/EC to indicate that it must not be disposed of with your other household waste. Due to the presence of hazardous substances, mixtures or components, electrical and electronic devices that are not subject to selective sorting are potentially dangerous to the environment and human health. Please check your local city office or waste disposal service for the return and recycling of this product.

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Ultimate Products UK Ltd.,
Ultimate Products Europe Ltd.,