
User manual

Mechanical bathroom scale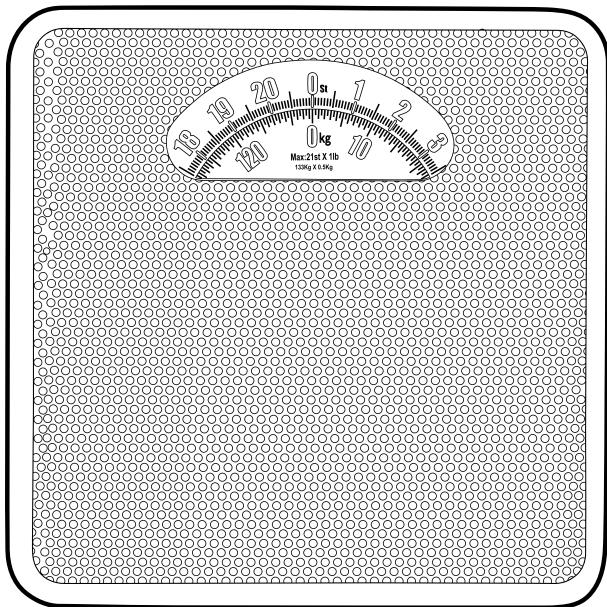

SAFETY INSTRUCTIONS

- This product is not a toy.
- Children must be supervised to ensure that they do not handle or play with the product.
- If any part of the product is malfunctioning or if it has been dropped or damaged, cease using the product immediately to avoid potential injury.
- Do not insert fingers into any openings or inside the product.
- Do not place, store or use the product near direct sunlight or direct heat sources or in a damp/high humidity environment.
- This product is a high precision device; handle it with care.
- This product is intended for domestic use only. It should not be used for commercial purposes.

Care and maintenance

Wipe the exterior of the scale with a soft, damp cloth and allow to dry thoroughly.

Never immerse the scale in water or any other liquid.

Do not use harsh or abrasive cleaning detergents or scourers to clean the scale, as this could cause damage.

Using the scale

STEP 1: Place the scale on a firm, flat surface and set the scale to '0' using the serrated wheel.

STEP 2: Gently tap on the scale to settle the internal mechanism, readjusting to '0' using the serrated wheel if necessary.

STEP 3: Step onto the scale, standing as centrally and as still as possible, with feet evenly spaced.

NOTE: For consistency, always weigh on the same scale placed on the same floor surface. Do not compare weight readings from one scale to another as some differences will exist due to manufacturing tolerances. Placing the scale on a hard, even floor will ensure the greatest accuracy and consistency. It is recommended that readings are taken at the same time of day, preferably early evening and before a meal for the most consistent results.

CAUTION: To avoid tipping the scale when stepping on and off, always stand centrally on the platform and distribute weight evenly.

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Ultimate Products UK Ltd.,
Ultimate Products Europe Ltd.,