User manual

Scale

00 KG

SALTER

EN | Please retain instructions for future reference.

SAFETY INSTRUCTIONS

- This product is not a toy.
- Children must be supervised to ensure that they do not handle or play with the product.
- This product contains no user serviceable parts. Only a qualified electrician should attempt repairs.
- If any part of the appliance is malfunctioning or if it has been dropped or damaged, cease using the product immediately to avoid potential injury.
- Do not insert fingers into any openings or inside the product.
- Do not place, store or use the product near direct sunlight or direct heat sources or in a damp/high humidity environment.
- This product is a high precision device; handle it with care.
- This product is intended for domestic use only.

Care and maintenance

Wipe the exterior of the scale with a soft, damp cloth and allow it to dry thoroughly.
Never immerse the scale in water or any other liquid.
Do not use harsh or abrasive cleaning detergents or scourers to clean the scale, as this could cause damage.

Changing the unit

STEP 1: Press the centre of the glass platform to turn on the scale.

STEP 2: Select kg, st or lb weight mode using the switch on the underside of the scale. The scale will then turn off.

Setting the scale to zero

STEP 1: Position the scale on a flat, level surface.

STEP 2: Press the centre of the platform to turn on the scale. Make sure no weight is on the scale. '0.0' will be displayed.

STEP 3: The scale will switch off and is now ready for use.

NOTE: The zero-setting process must be repeated if the scale is moved. At all other times step straight on the scale.

Using the scale

STEP 1: Step on to the platform and stand still.

STEP 2: After 2–3 seconds the weight reading will be displayed.

STEP 3: Step off the platform; the weight reading will be displayed for several seconds before the scale turns off.

NOTE: It is recommended that readings are taken at the same time of day, preferably early evening and before a meal for the most consistent results.

Troubleshooting

If the scale is not working properly:

Check that the correct unit is selected.
Make sure the scale is on a firm, flat surface and not touching a wall.
Repeat the zero-setting process.

Warning indicators

Err – Weight exceeds maximum capacity.

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Skärm: LCD

Enheter: kg/st/lbs

EN | Disposal of Waste Batteries and Electrical and Electronic Equipment

This product is labelled with this crossed out wheel bin symbol in accordance with European Directive 2012/19/EC to indicate that it must not be disposed of with your other household waste. Due to the presence of hazardous substances, mixtures or components, electrical and electronic devices that are not subject to selective sorting are potentially dangerous to the environment and human health. Please check your local city office or waste disposal service for the return and recycling of this product.

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Ultimate Products UK Ltd.,
Ultimate Products Europe Ltd.,