


### Step 1

Please confirm your model number is:

BPA - 9301

The model no. is located on the base of your product

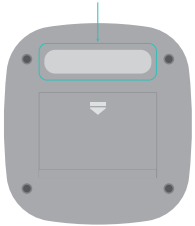


### Step 2


Please choose a user number for your measurements




Ensure that the batteries are correctly fitted and Bluetooth is enabled on your mobile device

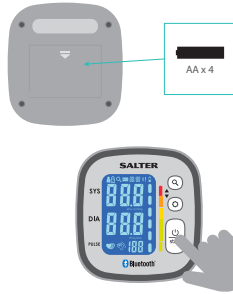

### Step 3

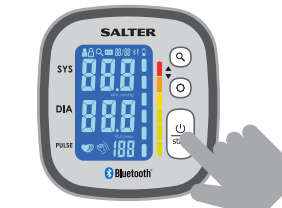





Ensure the device is turned off  
Press and hold the  button until the animation appears

### Step 4

Select User Number



With the device turned off, press and hold the  button until the user no. icons flash (). Then press the  button to cycle between users 1, 2 and guest, press  to confirm your user number

Guest readings are not saved

### Step 5



Place the cuff around your arm following the printed directions and then press the start button

Ensure you sit still with your arm below your heart for the entire procedure


### Step 6




At the end of the measurement, if your app is open then your data will be synced automatically

If your monitor is unable to connect, then your data will be saved to your monitor until a connection is available

### Step 7

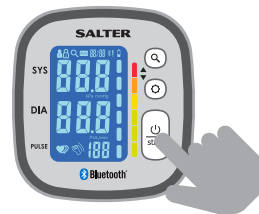

You can sync your saved data at any time by opening your app and then pressing the  button on your blood pressure monitor