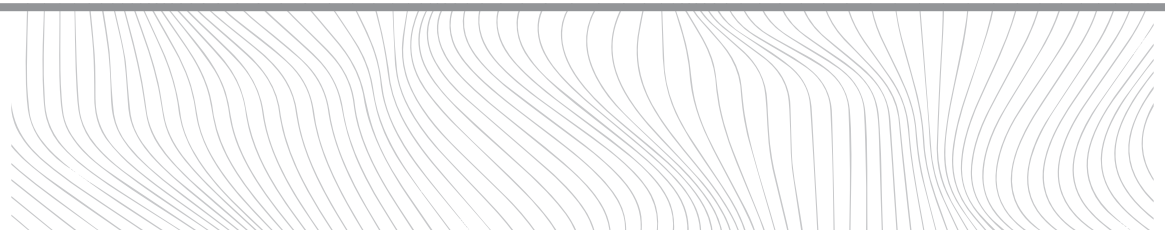
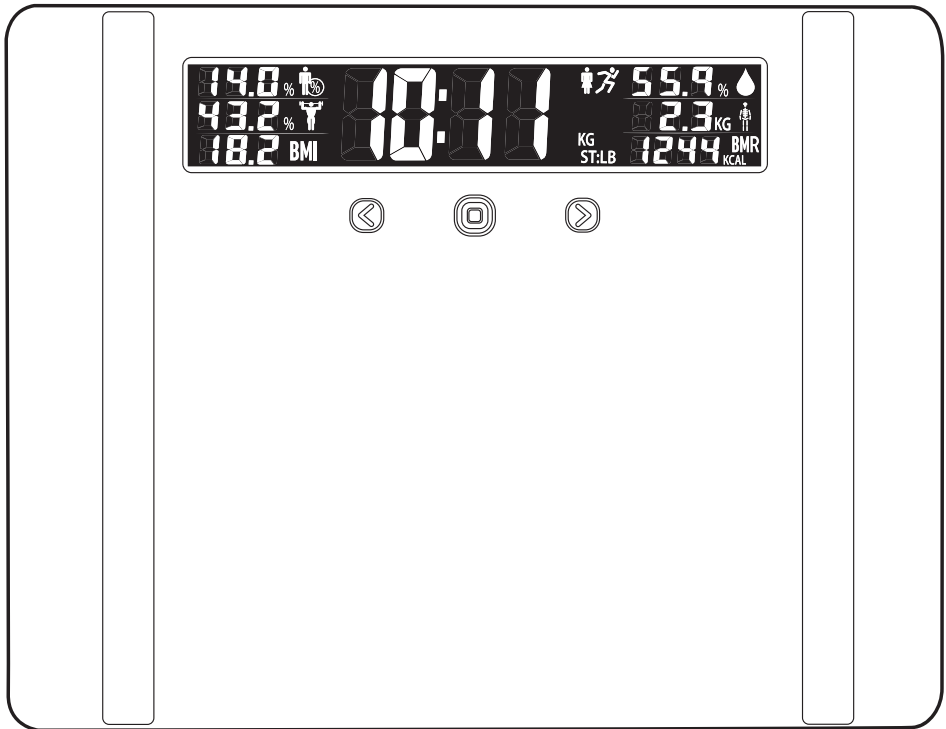
User manual

Extra wide analyser scale

EN | Please retain instructions for future reference.

SAFETY INSTRUCTIONS

BATTERY WARNING: KEEP OUT OF THE REACH OF CHILDREN.

- This product is not a toy.
- Children must be supervised to ensure that they do not handle or play with the product.
- This product contains no user serviceable parts. Only a qualified electrician should attempt repairs.
- If any part of the appliance is malfunctioning or if it has been dropped or damaged, cease using the product immediately to avoid potential injury.
- Do not insert fingers into any openings or inside the product.
- Do not place, store or use the product near direct sunlight or direct heat sources or in a damp/high humidity environment.
- This product is a high precision device; handle it with care.
- This product is intended for domestic use only.

CAUTION: Do not use the analyser function if you have a pacemaker or other medical device fitted. If in doubt, consult your doctor.

BATTERY SAFETY

- This product is supplied with 4 x AAA batteries. Only batteries of the same or equivalent type as those recommended should be used with this product.
- Batteries are to be inserted with the correct polarity.
- Exhausted batteries should be removed from the product to avoid leakage.

Care and maintenance

Wipe the exterior of the scale with a soft, damp cloth and allow it to dry thoroughly.

Never immerse the scale in water or any other liquid.

Do not use harsh or abrasive cleaning detergents or scourers to clean the scale, as this could cause damage.

Before first use

Remove the plastic isolator tab from the battery compartment (if present) prior to use.

Make sure the batteries are inserted following the correct polarity. Position the scale on a firm, flat surface.

For use on carpet attach enclosed carpet feet.

Changing the unit

STEP 1: Press the centre of the glass platform to turn on the scale.

STEP 2: Select kg, st or lb weight mode by pressing '>' or '<'. The scale will then turn off after approx. 1 minute.

Zero setting the scale

STEP 1: Position the scale on a firm, flat surface.

STEP 2: Press the centre of the platform to turn on the scale. Make sure no weight is on the scale. '0.0/ZERO' will be displayed.

STEP 3: The scale will switch off and is now ready for use.

NOTE: The zero-setting process must be repeated if the scale is moved. At all other times step straight onto the scale.

Using the scale to take weight readings

STEP 1: Step on to the platform and stand still.

STEP 2: After 2–3 seconds the weight reading will be displayed.

STEP 3: Step off the platform; the weight reading will be displayed for approx. 1 minute before the scale turns off.

Personal data entry

STEP 1: Press the centre of the platform to turn on the scale.

STEP 2: Press '⊙' to enter analyser mode.

STEP 3: Select a user number by pressing '>' or '<'. Press '⊙' to confirm the selection.

STEP 4: The male or female symbol will flash. Press '>' or '<' to select male/female. Select athlete mode by pressing '>' or '<' until both the male/female and athlete symbols are displayed. Press '⊙' to confirm.

STEP 5: The height display will flash. Press '>' or '<' to select the correct height, then press '⊙' to confirm.

STEP 6: The age display will flash. Press '>' or '<' to select the correct age, then press '⊙' to confirm.

STEP 7: The display will show the selected settings, then '0.0/ZERO' will be displayed. The memory is set.

STEP 8: Repeat procedure for another user (up to 10 times), or to change user details.

Athlete mode: An athlete is defined as a person who is involved in intense physical activity of approximately 12 hours a week and who has a resting heartbeat of approximately 60 beats per minute or less.

Using the scale

This scale uses BIA (Bio Impedance Analysis) technology which passes a small electrical impulse through the body to distinguish fat from lean tissue. The electrical impulse cannot be felt and is perfectly safe. Contact with the body is made via stainless steel pads beneath the glass platform.

NOTE: During pregnancy a woman's body composition changes considerably to support the developing child. Under these circumstances, body fat percentage readings could be inaccurate and misleading. Pregnant women should therefore only use the weight function.

For more information, visit salter.com/understanding-your-measurements.

STEP 1: Position the scale on a firm, flat surface.

STEP 2: Press the centre of the platform to turn on the scale.

STEP 3: Press '⊙' to enter analyser mode.

STEP 4: Select user number by using '>' or '<'. The LCD display will show the saved data for that user, then '0.0/ZERO' will be displayed.

STEP 5: With bare feet, step onto the platform and stand still. Feet must be in contact with the metal pads on the platform and evenly spaced.

STEP 6: After approx. 2–3 seconds, the weight reading will be displayed.

STEP 7: Remain standing on the scale while the other readings are taken.

STEP 8: Results will be displayed.

Troubleshooting

If the scale is not working properly:

Check the batteries are fitted correctly.

Check that the correct unit is selected.

Make sure the scale is on a hard, flat surface and not touching a wall.

Repeat the zero-setting process.

Warning indicators

Lo – Replace battery.

Lr – Weight exceeds maximum capacity.

Err2 – Out of range, incorrect operation or poor feet contact.

Specifications

Product code: SA00576
Display: LCD
Battery: 4 x 1.5 V AAA batteries (included)
Units: kg/st/lb

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EN Disposal of Waste Batteries and Electrical and Electronic Equipment

This symbol on the product, its batteries or its packaging means that this product and any batteries it contains must not be disposed of with household waste. Instead, it is the user's responsibility to hand this over to an applicable collection point for the recycling of batteries and electrical and electronic equipment. This separate collection and recycling will help to conserve natural resources and prevent potential negative consequences for human health and the environment due to the possible presence of hazardous substances in batteries and electrical and electronic equipment, which could be caused by inappropriate disposal. Some retailers provide take-back services which allow the user to return exhausted equipment for appropriate disposal. It is the user's responsibility to delete any data on electrical and electronic equipment prior to disposal. For more information about where to drop batteries, electrical and electronic waste off, please contact the local city/municipality office, household waste disposal service, or the retailer.

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Ultimate Products UK Ltd.,
Ultimate Products Europe Ltd.,