
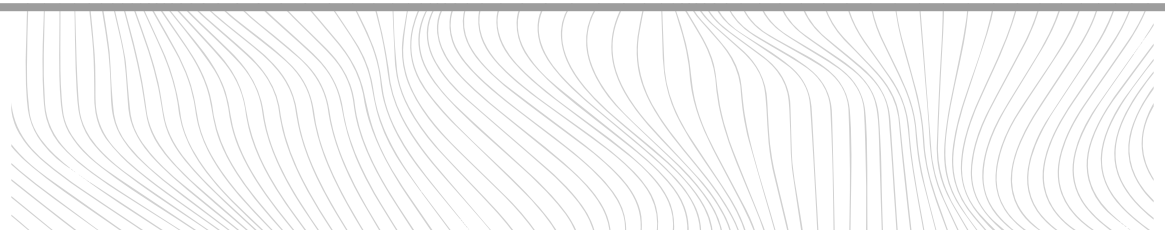
User manual

Smart scale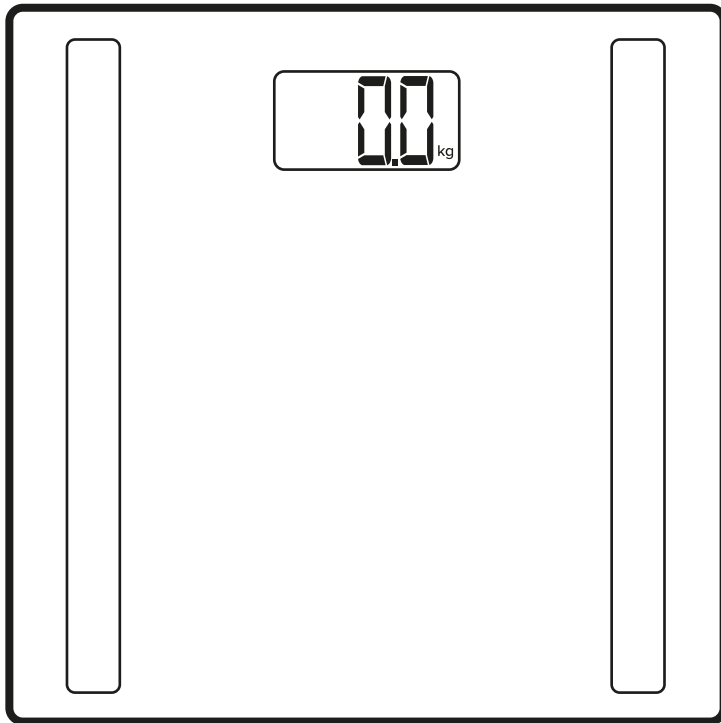

EN | Please retain instructions for future reference.

SAFETY INSTRUCTIONS

BATTERY WARNING: KEEP OUT OF THE REACH OF CHILDREN.

- This product is not a toy.
- Children must be supervised to ensure that they do not handle or play with the product.
- This product contains no user serviceable parts. Only a qualified electrician should attempt repairs.
- If any part of the appliance is malfunctioning or if it has been dropped or damaged, cease using the product immediately to avoid potential injury.
- Do not insert fingers into any openings or inside the product.
- Do not place, store or use the product near direct sunlight or direct heat sources or in a damp/high humidity environment.
- This product is a high precision device; handle it with care.
- This product is intended for domestic use only.

CAUTION: The user should not use the analyser function if they have a pacemaker or other medical device fitted. If in doubt, consult a doctor.

BATTERY SAFETY

- This product is supplied with 2 x AAA batteries. Only batteries of the same or equivalent type as those recommended should be used with this product.
- Batteries are to be inserted with the correct polarity.
- Exhausted batteries should be removed from the product to avoid leakage.

Care and maintenance

Wipe the exterior of the scale with a soft, damp cloth and allow it to dry thoroughly.

Never immerse the scale in water or any other liquid.

Do not use harsh or abrasive cleaning detergents or scourers to clean the scale, as this could cause damage.

Before first use

Remove the plastic isolator tab from the battery compartment (if present) prior to use.

Make sure the batteries are inserted following the correct polarity.

Position the scale on a firm flat surface.

Switching on the scale

STEP 1: Position the scale on a firm, flat surface.

STEP 2: Press the centre of the platform and make sure no weight is on the scale. **'0.0/ZERO'** will be displayed.

STEP 3: To select a weight unit (kg, st or lb) press **'UNIT'** on the underside of the scale.

STEP 4: The scale will switch off and is now ready for use.

NOTE: The setup process must be repeated if the scale is moved. At all other times, step straight on to the scale.

Using the scale to take weight readings

STEP 1: Step on to the platform and stand still.

STEP 2: After 2–3 seconds the weight reading will be displayed.

STEP 3: Step off the platform; the weight reading will be displayed for several seconds before the scale turns off.

Pairing a mobile device

STEP 1: Download and install the Salter Health app on a mobile device.

STEP 2: Turn on Bluetooth in the settings menu on the mobile device.

STEP 3: Open the Salter Health app and follow the on-screen instructions to set up a profile.

STEP 4: Once account is set up, navigate to the settings menu in the app.

STEP 5: Select **'Add Device'**.

STEP 6: Select **'SA00656 Smart Scale'** from the list of devices.

STEP 7: Follow the instructions shown on the mobile device to finish setup.

STEP 8: Repeat steps 1–7 for each user and mobile device.

NOTE: Each user number can only be paired with one mobile device. Results will be available on the paired device only.

Personal data entry

STEP 1: Open the Salter Health app and navigate to the settings menu in the app.

STEP 2: Select **'My Settings'** and input all personal data.

STEP 3: Repeat for a second user, or to change user details.

NOTE: If the age is set between 6–12 years, child mode will automatically be selected. In child mode only weight, body fat and BMI are displayed.

Athlete mode: If the age is set between 15–50 years, athlete mode is available via **'My Settings'** in the app. An athlete is defined as a person who is involved in intense physical activity of approximately 12 hours a week and who has a resting heartbeat of approximately 60 beats per minute or less. In athlete mode only weight, body fat and BMI are displayed if the age is 15–17 years.

Using the scale

This scale uses BIA (Bio Impedance Analysis) technology which passes a small electrical impulse through the body to distinguish fat from lean tissue. The electrical impulse cannot be felt and is perfectly safe. Contact with the body is made via stainless steel pads beneath the glass platform. For more information visit salter.com/understanding-your-measurements/.

STEP 1: Position the scale on a firm, flat surface.

STEP 2: Press the centre of the platform and make sure no weight is on the scale. **'0.0'** will be displayed and the Bluetooth icon will flash.

STEP 3: Open the Salter Health app to connect the paired device to the scale. The Bluetooth icon will remain constant on the scale display once connected.

STEP 4: Ensure the desired unit is selected via **'My Settings'** in the app. Exit **'My Settings'** to update the unit on the scale.

STEP 5: With bare feet, step onto the platform and stand still. Feet must be in contact with the metal pads on the platform and evenly spaced.

STEP 6: After 2–3 seconds, the weight will be displayed.

STEP 7: Remain standing on the scale while the other readings are taken.

STEP 8: **'New Reading'** followed by the date will appear on the mobile device to confirm the measurement has been recorded.

Warning indicators

Lo – Replace battery.

Err – Weight exceeds maximum capacity.

Err2 – Out of range, incorrect operation or poor feet contact.

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LCD: 2 batterie AAA da 1,5 V (incluse)

Unità: kg/st/lb

EN | Disposal of Waste Batteries and Electrical and Electronic Equipment

This symbol on the product, its batteries or its packaging means that this product and any batteries it contains must not be disposed of with household waste. Instead, it is the user's responsibility to hand this over to an applicable collection point for the recycling of batteries and electrical and electronic equipment. This separate collection and recycling will help to conserve natural resources and prevent potential negative consequences for human health and the environment due to the possible presence of hazardous substances in batteries and electrical and electronic equipment, which could be caused by inappropriate disposal. Some retailers provide take-back services which allow the user to return exhausted equipment for appropriate disposal. **It is the user's responsibility to delete any data on electrical and electronic equipment prior to disposal.** For more information about where to drop batteries, electrical and electronic waste off, please contact the local city/municipality office, household waste disposal service, or the retailer.

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Ultimate Products UK Ltd.,
Ultimate Products Europe Ltd.,