SALTER®

SINCE 1760

Dashboard Analyser Scale

Instruction Manual

EN | Please retain instructions for future reference.

Safety Instructions

BATTERY WARNING: KEEP OUT OF THE REACH OF CHILDREN

This product contains no user serviceable parts. Only a qualified electrician should attempt repairs.

Do not place, store or use the product near direct sunlight or direct heat sources or in a damp/ high humidity environment.

This product is a high precision device; handle it with care.

This product is intended for domestic use only.

CAUTION: Do not use the analyser function if you have a pacemaker or other medical device fitted. If in doubt, consult your doctor.

Battery Safety

This product is supplied with 3 x AAA batteries. Only batteries of the same or equivalent type as those recommended should be used with this product.

Batteries are to be inserted with the correct polarity.

Exhausted batteries should be removed from the product to avoid leakage.

Care and Maintenance

Wipe the exterior of the scale with a soft, damp cloth and allow it to dry thoroughly. Never immerse the scale in water or any other liquid. Do not use harsh or abrasive cleaning detergents or scourers to clean the scale, as this could cause damage.

Instructions for Use

Before First Use

Remove the plastic isolator tab from the battery compartment (if present) prior to use; this will power up the scale. Make sure the batteries are inserted following the correct polarity. If the scale is being used on a carpeted floor attach the included carpet feet.

Switching On the Body Analyser Scale

This scale uses BIA (Bio Impedance Analysis) technology which passes a small electrical impulse through the body to distinguish fat from lean tissue. The electrical impulse cannot be felt and is perfectly safe. Contact with the body is made via stainless steel pads beneath the glass platform. For more information visit <https://salter.com/understanding-your-measurements/>.

STEP 1: Press the centre of the glass platform and make sure no weight is on the scale. '0.0/ZERO' will be displayed.

STEP 2: Select kg, st or lb weight mode by pressing [**>**] or [**<**] buttons.

STEP 3: The scale will switch off and is now ready for use.

NOTE: The initialisation process must be repeated if the scale is moved.
At all other times step straight on the scale.

Using the Body Analyser for a Weight Reading Only

STEP 1: Step on to the platform and stand still.

STEP 2: After 2–3 Seconds the weight reading will be displayed.

STEP 3: Step off, the weight will be displayed for a few seconds and then the scale will turn off.

Setting User Memory

STEP 1: Press on the platform centre with your foot.

STEP 2: Press the [**0**] button, a user number will flash.

STEP 3: Once the user number is flashing, select by pressing [**>**] or [**<**]. Press [**0**] to confirm the selection.

STEP 4: The male or female symbol will flash. Press [**>**] to select female, male, female athlete or male athlete, then press [**0**].

STEP 5: The height display will flash next. Press [**>**] or [**<**] as necessary to set user height, then press [**0**].

STEP 6: To set user age, wait until the age display flashes. Press [**>**] and [**<**] to set age then press [**0**] to confirm.

STEP 7: '0.0/ZERO' will be displayed. To add another user or update the memorised data, repeat steps 1–6.

ATHLETE MODE: An athlete is defined as a person who is involved in intense physical activity of approximately 12 hours per week and who has a resting heart rate of approximately 60 beats per minute or less.

NOTE: Athlete Mode will only function if user age is set between 15 and 50 years.

Using the Body Analyser Scale

STEP 1: Position the scale on a flat, level surface. Press on the platform centre with your foot.

STEP 2: Press the [**0**] button. Select user number by pressing [**>**] or [**<**].

STEP 3: The LCD display will show the saved data for that user, followed by a zero reading.

STEP 4: With bare feet, step onto the glass platform and stand still. Feet must be in contact with the metal pads on the platform.

STEP 5: The weight reading will be followed by body fat percentage, total body water percentage, BMI, bone mass, muscle mass and BMR.

NOTE: For consistency, always weigh on the same scale placed on the same floor surface.

Do not compare weight readings from one scale to another as some differences will exist due to manufacturing tolerances. Placing the scale on a hard, even floor will ensure the greatest accuracy and consistency. It is recommended that readings are taken at the same time of day, preferably early evening and before a meal for the most consistent results.

The scale rounds up or down to the nearest increment. If weight is measured twice and the readings differ by one increment, the weight lies between the two readings.

Troubleshooting

'Err2' is showing on the LCD display when taking body fat and body water measurements.

The user does not have bare feet. Remove shoes and socks before using the scale.

The condition of the user's skin is affecting the reading. Wipe feet with a damp cloth, leaving them slightly moist before stepping onto the scale.

Warning indicators

Lo – replace battery.

0-Ld – weight exceeds maximum capacity.

ERR – unstable weight. Stand still.

Err2 – out of range, incorrect operation or poor feet contact.

Specifications

Product code: SA00325

Display: LCD

Battery: 3 x 1.5 V AAA batteries (included)

Unit of measurement: kg/st/lb

Disposal of Waste Batteries and Electrical and Electronic Equipment

 This symbol on the product, its batteries or its packaging means that this product and any batteries it contains must not be disposed of with household waste. Instead, it is the user's responsibility to hand this over to an applicable collection point for the recycling of batteries and electrical and electronic equipment. This separate collection and recycling will help to conserve natural resources and prevent potential negative consequences for human health and the environment due to the possible presence of hazardous substances in batteries and electrical and electronic equipment, which could be caused by inappropriate disposal. Some retailers provide take-back services which allow the user to return exhausted equipment for appropriate disposal. It is the user's responsibility to delete any data on electrical and electronic equipment prior to disposal. For more information about where to drop batteries, electrical and electronic waste off, please contact the local city/municipality office, household waste disposal service, or the retailer.

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EN If this product does not reach you in an acceptable condition, please contact our Customer Services Department at www.salter.com. Please have your delivery note to hand as details from it will be required. If you wish to return this product, please return it to the retailer from where it was purchased with your receipt (subject to their terms and conditions).

To be eligible for the extended guarantee, go to guarantee.upgs.com/salter and register your product within 30 days of purchase.

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©Salter trademark. All rights reserved.
UP Global Sourcing UK Ltd.,