
# User manual

Electronic scale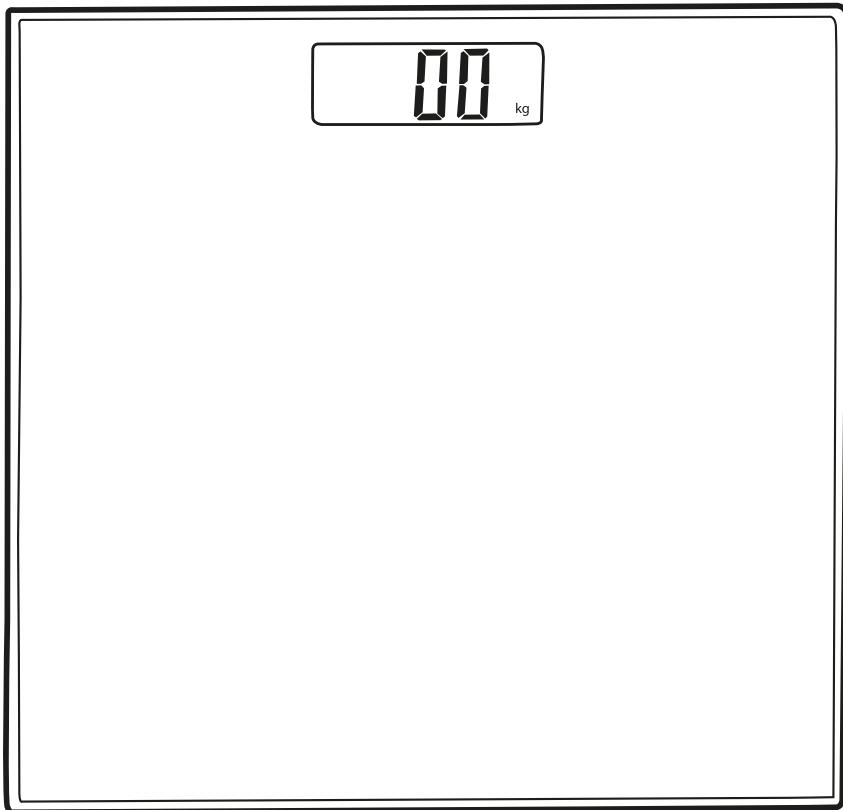

**EN |** Please retain instructions for future reference.

## SAFETY INSTRUCTIONS


### BATTERY WARNING:

#### KEEP OUT OF THE REACH OF CHILDREN.

- This product is not a toy.
- This product contains no user serviceable parts. Only a qualified electrician should attempt repairs.
- If any part of the appliance is malfunctioning or if it has been dropped or damaged, cease using the product immediately to avoid potential injury.
- Do not place, store or use the product near direct sunlight or direct heat sources or in a damp/high humidity environment.
- This product is a high precision device; handle it with care.
- This product is intended for domestic use only.

### BATTERY SAFETY

- This product is supplied with 1 x CR2032 battery. Only batteries of the same or equivalent type as those recommended should be used with this product.
- Batteries are to be inserted with the correct polarity.
- Exhausted batteries should be removed from the product to avoid leakage.

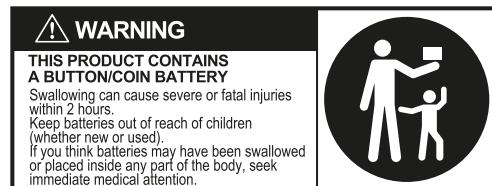**WARNING:** If the battery compartment does not close securely, stop using the product and keep it away from children.

**CAUTION:** Vigilance must be maintained with "flat" or spare button or coin batteries in the home and the products that contain them.

There are no obvious specific symptoms when a button or coin battery is stuck in a child's oesophagus (food pipe). However, the child might exhibit the following symptoms:

- Coughing, gagging or drooling
- Appear to have a stomach upset or virus
- Vomiting
- Pointing to their throat or stomach
- Pain in their abdomen, chest or throat
- Tiredness and lethargy
- Be quieter or more clingy than usual
- Loss or reduction of appetite
- Refusal or inability to eat solid foods

These sorts of symptoms may fluctuate.

**A specific symptom of button or coin battery ingestion is vomiting fresh (bright red) blood. Seek immediate medical attention if this occurs.**

### Care and maintenance

Wipe the exterior of the scale with a soft, damp cloth and allow it to dry thoroughly.

Never immerse the scale in water or any other liquid.

Do not use harsh or abrasive cleaning detergents or scourers to clean the scale, as this could cause damage.

## Instructions for use

### Before first use

Loosen the battery safety screw and open the battery compartment on the underside of the scale.

Remove the plastic isolator tab from the battery compartment (if present) prior to use.

Check that the battery is inserted following the correct polarity. Close the battery compartment and tighten the battery compartment screw.

Position the scale on a firm flat surface.

For use on carpet, attach the enclosed carpet feet.

### Changing the Unit

**STEP 1:** Press the centre of the platform to turn on the scale.

**STEP 2:** Select kg, st or lb weight mode using the switch on the underside of the scale. The scale will then turn off.

### Setting the scale to zero

**STEP 1:** Press the centre of the glass platform and make sure no weight is on the scale. **[0.00/ZERO]** will be displayed.

**STEP 2:** The scale will switch off and is now ready for use.

**NOTE:** The zero-setting process must be repeated if the scale is moved. At all other times step straight on the scale.

### Using the electronic bathroom scale

**STEP 1:** Step on to the platform and stand still.

**STEP 2:** After 2–3 seconds the weight reading will be displayed.

**STEP 3:** Step off the platform; the weight reading will be displayed for several seconds before the scale turns off.

**NOTE:** It is recommended that readings are taken at the same time of day, preferably early evening and before a meal for the most consistent results.

### Troubleshooting

#### If the scale is not working properly:

Check the battery is fitted correctly.

Check that the correct unit is selected.

Make sure the scale is on a hard, flat surface and not touching a wall.

Repeat the zero-setting process.

### Warning indicators

**Lo** – Replace battery.

**EE / O-Ld** – Weight exceeds maximum capacity.

**Err** – Unstable weight. Stand still.

### Specifications

Product code: GENBATH1

Display: LCD

Battery: 1 x 3 V CR2032 button cell battery (included)

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Enheter: kg/st/lbs

#### EN | Disposal of Waste Batteries and Electrical and Electronic Equipment

This symbol on the product, its batteries or its packaging means that this product and any batteries it contains must not be disposed of with household waste. Instead, it is the user's responsibility to hand this over to an applicable collection point for the recycling of batteries and electrical and electronic equipment. This separate collection and recycling will help to conserve natural resources and prevent potential negative consequences for human health and the environment due to the possible presence of hazardous substances in batteries and electrical and electronic equipment, which could be caused by inappropriate disposal. Some retailers provide take-back services which allow the user to return exhausted equipment for appropriate disposal. It is the user's responsibility to delete any data on electrical and electronic equipment prior to disposal. For more information about where to drop batteries, electrical and electronic waste off, please contact the local city/municipality office, household waste disposal service, or the retailer.

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Ultimate Products UK Ltd.,
Ultimate Products Europe Ltd.,