
# User manual

Compact analyser scale

To register for your extended guarantee,  
scan the QR code or visit  
[guarantee.upgs.com/salter](http://guarantee.upgs.com/salter).  
For any further queries, visit [www.salter.com](http://www.salter.com).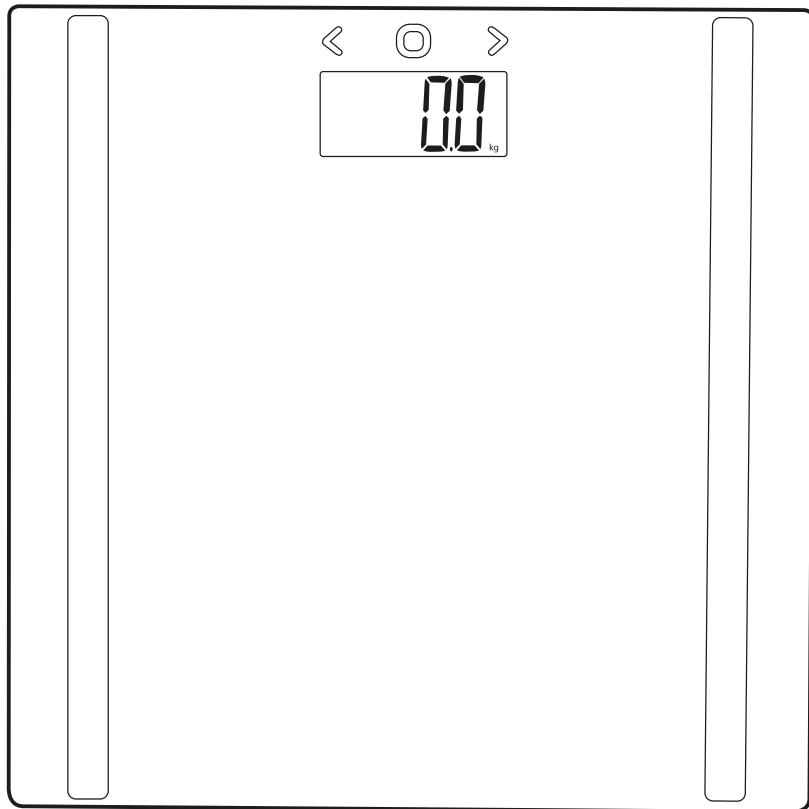

**SALTER**  
1760

**EN |** Please retain instructions for future reference.

## SAFETY INSTRUCTIONS

### BATTERY WARNING:

#### KEEP OUT OF THE REACH OF CHILDREN.

- This product is not a toy.
- This product contains no user serviceable parts. Only a qualified electrician should attempt repairs.
- If any part of the appliance is malfunctioning or if it has been dropped or damaged, cease using the product immediately to avoid potential injury.
- Do not place, store or use the product near direct sunlight or direct heat sources or in a damp/high humidity environment.
- This product is a high precision device; handle it with care.
- This product is intended for domestic use only.

**CAUTION:** The user should not use the analyser function if they have a pacemaker or other medical device fitted. If in doubt, consult a doctor.

### BATTERY SAFETY

- This product is supplied with 2 x AAA batteries. Only batteries of the same or equivalent type as those recommended should be used with this product.
- Batteries are to be inserted with the correct polarity.
- Exhausted batteries should be removed from the product to avoid leakage.

### Care and maintenance

Wipe the exterior of the scale with a soft, damp cloth and allow it to dry thoroughly.  
Never immerse the scale in water or any other liquid.  
Do not use harsh or abrasive cleaning detergents or scourers to clean the scale, as this could cause damage.

### Before first use

Make sure the batteries are inserted following the correct polarity.  
Position the scale on a firm flat surface.  
For use on carpet attach enclosed carpet feet.

### Changing the unit

**STEP 1:** Press the centre of the glass platform to turn on the scale.  
**STEP 2:** Select kg, st or lb weight mode by pressing [**UNIT**] on the base of the scale. The scale will then turn off.

### Setting the scale to zero

**STEP 1:** Position the scale on a flat, level surface.  
**STEP 2:** Press the centre of the platform to turn on the scale, then make sure no weight is on the scale. [**0.0/ZERO**] will be displayed.

**STEP 3:** The scale will switch off and is now ready for use.

**NOTE:** THE ZERO-SETTING PROCESS MUST BE REPEATED IF THE SCALE IS MOVED. AT ALL OTHER TIMES STEP STRAIGHT ONTO THE SCALE.

### Using the scale to take weight readings

**STEP 1:** Step on to the platform and stand still.  
**STEP 2:** After 2–3 seconds the weight reading will be displayed.  
**STEP 3:** Step off the platform; the weight reading will be displayed for several seconds before the scale turns off.

### Setting user memory

- STEP 1:** Press the select button to turn the scale on.
- STEP 2:** While the user number is flashing, select a user number by pressing the arrow buttons. Press the select button to confirm the user number.
- STEP 3:** The male [♂] or female [♀] symbol will flash. Press the arrow buttons to select male [♂] or female [♀]. Press the select button to confirm.
- STEP 4:** The age display will flash. Press the arrow buttons to select the correct age, then press the select button to confirm.
- STEP 5:** The height display will flash. Press the arrow buttons to select the correct height, then press the select button to confirm.
- STEP 6:** The athlete symbol will flash. Press the arrow buttons to turn athlete mode on/off, then press the select button to confirm.
- STEP 7:** The scale will display the settings then switch off. The memory is set.
- STEP 8:** Repeat the above procedure to add a second user, or to change user details.

**NOTE:** If age is set between 6–12 years, child mode will automatically be selected. In child mode only weight, body fat and body water are displayed.

The buttons are located under the glass on the top edge of the scale.

**ATHLETE MODE:** If the age is set between 15–50 years athlete mode is available. An athlete is defined as a person who is involved in intense physical activity of approximately 12 hours a week and who has a resting heartbeat of approximately 60 beats per minute or less.

### Using the scale

This scale uses BIA (Bio Impedance Analysis) technology which passes a small electrical impulse through the body to distinguish fat from lean tissue. The electrical impulse cannot be felt and is perfectly safe. Contact with the body is made via stainless steel pads beneath the glass platform.

**NOTE:** During pregnancy a woman's body composition changes considerably to support the developing child. Under these circumstances, body fat percentage readings could be inaccurate and misleading. Pregnant women should therefore only use the weight function. For more information, visit [salter.com/understanding-your-measurements](http://salter.com/understanding-your-measurements).

- STEP 1:** Position the scale on a flat, level surface. Press the select button to turn the scale on.
- STEP 2:** Select user number by using the arrow buttons.
- STEP 3:** The LCD display will show the saved data for that user, followed by a zero reading.
- STEP 4:** With bare feet, step onto the platform and stand still. Make sure feet are evenly spaced on the platform.
- STEP 5:** After 2–3 seconds, the weight reading will be displayed.
- STEP 6:** Remain standing on the scale while the other readings are taken.
- STEP 7:** The weight will be displayed followed by body fat %, body water % and BMI.

## Troubleshooting

### If the scale is not working properly:

Check the battery is fitted correctly.

Check that the correct unit is selected.

Make sure the scale is on a hard, flat surface and not touching a wall.

Repeat the zero-setting process.

### Warning indicators

**Lo** – Replace battery.

**O-Ld** – Weight exceeds maximum capacity.

**Err2** – Out of range, incorrect operation or poor feet contact.

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## Guarantee

**EN** If this product does not reach you in an acceptable condition, please contact our Customer Services Department at [www.salter.com](http://www.salter.com). Please have your delivery note to hand as details from it will be required. If you wish to return this product, please return it to the retailer from where it was purchased with your receipt (subject to their terms and conditions). To be eligible for the extended guarantee, go to [guarantee.upgs.com/salter](http://guarantee.upgs.com/salter) and register your product within 30 days of purchase.

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#### EN | Disposal of Waste Batteries and Electrical and Electronic Equipment

This symbol on the product, its batteries or its packaging means that this product and any batteries it contains must not be disposed of with household waste. Instead, it is the user's responsibility to hand this over to an applicable collection point for the recycling of batteries and electrical and electronic equipment. This separate collection and recycling will help to conserve natural resources and prevent potential negative consequences for human health and the environment due to the possible presence of hazardous substances in batteries and electrical and electronic equipment, which could be caused by inappropriate disposal. Some retailers provide take-back services which allow the user to return exhausted equipment for appropriate disposal. It is the user's responsibility to delete any data on electrical and electronic equipment prior to disposal. For more information about where to drop batteries, electrical and electronic waste off, please contact the local city/municipality office, household waste disposal service, or the retailer.

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©Salter trademark. All rights reserved.
Ultimate Products UK Ltd.,
Ultimate Products Europe Ltd.,